

# CPAP & CPAP Supplies – Cleaning/Replacement Guide

## CPAP Accessories



Mask Frame + Cushion

Wipe down



Humidification Chamber

Wash with Antibacterial soap and water  
or  
A white vinegar and warm water solution



Heated Tube or Standard Tube.

Wash with Antibacterial soap and water  
or  
A white vinegar and warm water solution

\*Do not wipe  
items dry.

Place them in and  
area where they  
can air dry.\*

**Once a Week**

## Full Face Mask Replacment Frequency



Full Face Mask



Full Face Cushion

**Once a month**



CPAP Filters

**Every 2 weeks**

## Nasal Mask/Pillows Replacement Frequency



Nasal Mask/Nasal Pillows



Nasal Mask  
Cushion



Nasal Pillow  
Cushion



CPAP Filters

**Every 2 weeks**